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In case additional lubrication is needed, only lubricants are recommended which are recommended to be used together with condoms. Do not use any oil-based lubricants such as e.g. petroleum jelly, baby oil, body lotion, massage oil, butter or margarine as lubricant as these are deleterious to the integrity of the condom. Please attend that in case you parallel apply any prescription or over-the-counter medicines to the penis or the vagina, please consult a doctor or a pharmacist about the compatibility with the condom.

In case of slight irritations e.g. skin irritation or the like please also consult your doctor. In case that during use a tear or hole in the condom was found please contact a doctor within 72 h.

SICO condoms are a reliable means of protection against pregnancy and sexually transmitted diseases. No contraceptive can protect one hundred percent. More information see below: www.cpr-germany.de

Before any sexual contact condoms should be rolled over the erect penis, in order to protect against sexually transmitted diseases and pregnancy.

Remove the condom carefully with your fingers from the foil and treat it carefully in order to avoid damages by finger nails, jewellery etc.

In case the individual container is damaged, do not use it but use a new one which is not damaged.

Pinch the reservoir with the thumb and the index finger and place the condom on the erect penis.

Thus there will be no air cushion. Withdraw the prepuce for your security. Unroll the ring with the other hand completely. If during application there is a feeling that the condom is slipping off or is overstretched over the penis, please stop the application and check the condom because of danger of rupture. After ejaculation hold the condom at the ring and withdraw the penis with the condom.

Do not put any used condoms into the toilet. Please put used condoms into the waste.

Important! Please use the condom only once as otherwise there is a risk of infection and security features may be reduced.

Any user who knows about an allergy against Latex should not use any condoms from natural rubber latex as an overreaction or even an anaphylactic shock cannot be excluded.

Protect condoms against heat and direct sunlight.

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