DE

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EN

Additional Information Condoms:

Handle carefully, when removing from package avoid damage by fingernails, jewellery etc. If being damaged replace an individual container by a new one and discard the old one. Ask a doctor or pharmacist about compatibility of any topical medicines in possible contact with condom. If additional lubricant should be added, use only those recommended for condoms and never oily ones like petroleum jelly, baby oil, body lotion, massage oil, butter, margarine etc. Place condom on erect penis before and contact between penis and partner body occurs to assist preventing STIs or pregnancy. Stop if you feel condom slipping off or tightening excessively onto penis because of breakage danger. Withdraw penis soon after ejaculation by holding condom in place at penis base. Should a condom burst or leak during use seek medical assistance as soon as possible, at least within 72 h. Dispose used condom to waste, not to toilet. Don't use twice, reuse might increase risk of failure or infection. For vaginal use only. NR-Latex might cause allergic reaction incl. anaphylactic shock. MEIN KONDOM condoms are a reliable protection but no contraceptive can protect 100 % against pregnancy, AIDS and/or infectious diseases.

For Benzocaine condoms: Do not use in case of vaginal inflammations or other vaginal diseases or if you or your partner suffer from difficulties of breathing or gets blue lips after use.

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mpt med production and trading GmbH